

# OLIVE BEDSIDE TROLLEY



**⚠ CAUTION**

Read and understand all of the instructions and safety information in this manual before operating this product.



Yellow triangle with exclamation mark indicates potential minor injury to personnel or damage to equipment



Yellow triangle with lightning bolt indicates possible electric shock hazard

## USER ATTENTION

All employees who will use the Olive Bedside trolley must be briefed in its proper use and operation before using it. All employees who will utilize the Bedside trolley should be aware that reasonable precautions must be taken to ensure patient safety and the Bedside trolley's performance.

This product should only be used for the purpose for which it was created. Using this device in a way that is not recommended by Olive Health Care may void the equipment manufacturer's warranty. Olive Health Care disclaims all responsibility for the effects of using this product for purposes other than those for which it was intended. Misuse or alteration of a product can be dangerous. Olive Health Care disclaims all obligation for the results of product modifications, as well as the repercussions of using this product in combination with other products.



For the purpose of safety, wear appropriate PPE kit: mask, gloves, eye protection and gown



Use of unqualified persons may lead to injury to personnel and/or damage to equipment. Personnel using the Olive Bedside trolley must be educated on best practices, use, and maintenance of the product to avoid injury or damage. Repair and adjustments should be attempted by persons qualified as representatives of the manufacturer

## PRODUCT DESCRIPTION

Olive Endoscopy point-of-use processing solutions allow you to stay updated with industry standards while also making cleaning easier. The bedside cart allows ease of cleaning scopes and storing cleaning solutions. The trays are sturdy and have ample space for endoscopes to be coiled within determined guidelines while being cleaned. It can also assist in carrying-out a first check for correct functioning of the endoscope channels.


## INTENDED USE

This product is designed for healthcare workers to undertake point-of-use cleaning/pre-cleaning of flexible GI endoscopes immediately after procedure in the procedure room.

## GENERAL SPECIFICATIONS

Tray size: 555mm x 650 mm x 160 mm

Internal volume (tray): 460mm x 560mm x 150 mm

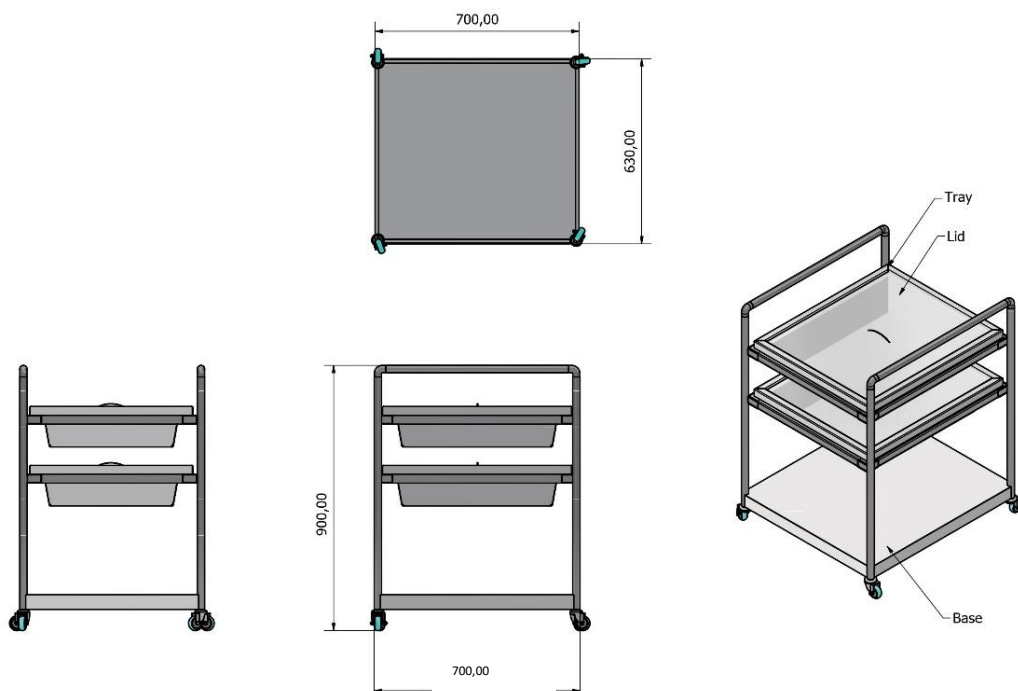

## LIST OF PARTS

Item Name	Model No.	Units
Body	OBTSSB01	1
Tray	OBTFT02	2
Wheel	OBTFWH04	4
Lid (optional)	OBTFL02	2
EZY Flush mounting (optional)	-	-

## MAINTENANCE AND BEST PRACTICES

**CAUTION:** The user is responsible for thoroughly cleaning and maintaining the bedside trolley to ensure optimum functioning.

**IMPORTANT:** To obtain the maximum benefit of the Bedside Trolley, certain procedures need to be followed:

1. Do NOT coil the universal cord of the endoscope with a diameter of less than 12 cm.
2. Attach the water resistant cap (as per manufacturer's IFU) BEFORE immersing the endoscope in fluids
3. Clean and disinfect the trays thoroughly on a regular cleaning protocol, or when the trays becomes soiled

## CLEANING, DISINFECTION & CARE

### Cleaning the Trays

Do not use abrasives, such as steel wool, wire brushes or scrapers on the stainless steel and PMMA fiber surface.

**CAUTION:** For plastic surfaces and acrylic plastic (PMMA Fiber) components, use a hospital-approved disinfectant. Avoid disinfectants that have glycol ethers as an ingredient. Do not use alcohol or ammonia.

### Cleaning Stainless Steel Body of Sink

1. Remove dirt, blood, and other organic soil.
2. Always clean the body of the trolley prior to disinfection to avoid compromising the disinfectant's effectiveness.
3. Dust first, then remove any stubborn deposits with a damp sponge.
4. Using a soft cloth, clean with a non-abrasive, neutral liquid detergent at a 10-20% solution. Use a stronger concentration in the presence of excessive dirt.
5. Lightly rinse before disinfection, being careful to minimize moisture in the sinks.
6. Avoid using bleach for cleaning or disinfecting your sink's surface, as bleach causes permanent damage through oxidation and staining.

Given below is a list of materials/ agents that are approved to be used to clean Olive Bedside Trolley:

1. Soft, lint-free cloth
2. Mild detergents
3. White vinegar in a spray bottle
4. Sodium bicarbonate (Baking Soda)
5. Cleaners approved for use on Stainless Steel/Galvanized Iron
6. Non-abrasive cleaning pads
7. Isopropyl alcohol
8. Soft bristle brushes
9. Distilled water (pH rating 7.0) solely or with mild detergent
10. Hospital-grade non-bleach detergents



## REPAIRS AND REPLACEMENT

Please contact the Olive Health Care support team if you have any concerns with component replacement or repairs

## WARRANTY

Olive Health Care warrants to the original purchaser of the Olive Bedside Trolley that defective products/components shall be repaired/replaced by Olive Health care, so long as owned by the original purchases with limitations set forth as under:

The customer must obtain a return authorization supplied by the customer service team of Olive Health Care. The product must be returned adequately packed so as to avoid damages during transit, with copy of original sales receipt and copy of the RMA authorization

The customer must pay freight charges to and from Olive Health Care manufacturing unit. Olive Health Care does not warrant any damage charges caused during transit. Customer shall be responsible for detailed inspection before signing the BOL

This warranty shall not apply to product(s)/components that have been subject to misuse, negligence, modification, alteration, lack of routine maintenance and/or misapplication, or products damaged due to servicing other than Olive Health Care representatives

This warranty shall not apply to components or parts not manufactured by Olive Health Care. Olive Health Care will pass on the original manufacturer's warranty to the original purchaser. Please consult the service team for warranty details of such components.

UNLESS EXPRESSLY SET FORTH, THERE SHALL BE NO OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, OF FITNESS AND/OR MERCHANTABILITY OR ANY OTHER WARRANTIES IMPLIED BY CUTOM,USAGE OR COURSE OF DEALING

Olive Health Care's liability in this agreement is limited to repair/ replacement of any product or component purchased by the original purchaser. Olive Health Care disclaims responsibility for any incidental or consequential damages claimed to have arisen from any defect in workmanship or materials. No representative of Olive Health Care has the authority to change or enlarge the above warranty agreement.

For further support,

Contact customer support team:

support@oliveendoscopy.com

Landline: 0484 403 8810