

# INSTRUCTION & OPERATING MANUAL

## OLIVE EZY FLUSH



Read and understand all of the instructions and safety information in this manual before operating this product.



Yellow triangle with exclamation mark indicates potential minor injury to personnel or damage to equipment



Yellow triangle with lightning bolt indicates possible electric shock hazard

## USER ATTENTION

All employees who will use the Olive EZY Flush must be briefed in its proper use and operation before using it. All employees who will utilise the Olive EZY Flush should be aware that reasonable precautions must be taken to ensure patient safety and the EZY Flush's performance.

This product should only be used for the purpose for which it was created. Using this device in a way that is not recommended by Olive Health Care may void the equipment manufacturer's warranty. Olive Health Care disclaims all responsibility for the effects of using this product for purposes other than those for which it was intended. Misuse or alteration of a product can be dangerous. Olive Health Care disclaims all obligation for the results of product modifications, as well as the repercussions of using this product in combination with other products.



For the purpose of operator safety, wear appropriate PPE kit: mask, gloves, eye protection and gown



Use of unqualified persons may lead to injury to personnel and/or damage to equipment. Personnel using the Olive EZY Flush must be educated on best practices, use, and maintenance of the product to avoid injury or damage. Repair and adjustments should be attempted by persons qualified as representatives of the manufacturer



Contact with live wire is possible. Ensure to switch off power supply before undertaking any repairs/inspection. Get in touch with Olive Health Care service team for repairs and electrical works.

## PRODUCT DESCRIPTION

Olive EZY Flush is designed to make flushing water and cleaning solutions through the channels of flexible, immersible endoscopes simpler. This device is intended for usage during the manual cleaning and disinfection phase of endoscope reprocessing before the endoscope is placed in an automated endoscope reprocessor (AER) for high-level disinfection.

## INTENDED USE

This product is designed for healthcare workers to utilise in order to flush internal channels of flexible endoscopes used in the treatment of gastrointestinal patients. Olive EZY Flush is also approved for use as an automated flushing device in other departments.

## INSTALLATION AND SET-UP

1. Counter Top Installation: Flat, level surface within reach of power outlet
2. Wall mounting: Olive EZY Flush should be installed no higher than 0.6 meters (1.5 feet) above the top of fluid solution in sink.
3. Attach Power Cord & power supply
4. Press power button ON until light illuminates
5. Attach the suction tube and the discharge tube to the front panel
6. Place the distal end of suction tube in water
7. Set the timer to required level

## GENERAL SPECIFICATIONS

Dimensions	165 (L) x 170 (W) x 225 (H)
Weight	5 Kg
Electrical Requirements	230 V AC/50Hz
Current Draw	< 1.0 Amperes
Flush Time(Adjustable)	0-99 seconds
Fluid Delivery Rate	2 LPM
Pump Pressure	1-2 Bar
Installation	Counter top/wall mountable
Accessories	EZY Flush Tubing Kit
Warranty Period	1 Year

### ACCESSORIES

Tubing kit: Suction tube and discharge tube, power adaptor (1)

### LIST OF PARTS

Item Name	Model No.	Units
Body	OEFSS01	1
Valve	OEFPV01	1
Timer	OEFET01	1
EZY Flush Pump	OEFDP01	1
Filter	OEFPF01	1
Check valve	OEFPCV01	1
Power cord	OEFA9V01	1
Lever connector	OEFPLC01	1
Suction tube	OEFPT1001	1
Discharge tube	OEFPT1002	1
Mounting kit	OEFSSMK01	1

### SYSTEM OPERATION

1. Connect the suction tube to the front panel and place the other end in clean water/detergent solution
2. Connect the discharge tube to the endoscope port opening
3. Use the buttons provided on the front panel to adjust/set timer
4. Press the button to flush water/detergent solution through channels
5. Upon completion, the screen reads '00'
6. Disconnect the tubes from the endoscope
7. Remove suction tube & place it on the sink counter

### MAINTENANCE AND BEST PRACTICES

**CAUTION:** The user is responsible for thoroughly cleaning and thorough leak testing of the endoscopes used in any medical procedures BEFORE flushing scope channels using Olive EZY Flush. Follow your institution's cleaning and disinfection procedures for reprocessing flexible endoscopes.

**IMPORTANT:** To obtain the maximum benefit of the EZY Flush, certain procedures must be followed:

1. Inspect Olive EZY Flush tubing for damage, leaks, or nicks
2. Replace the accessories if worn or damaged

### REPAIRS AND REPLACEMENT

Please contact the Olive Health Care support team if you have any concerns with component replacement or repairs

## WARRANTY

Olive Health Care warrants to the original purchaser of the Olive EZY Flush that defective products/components shall be repaired/replaced by Olive Health care, so long as owned by the original purchaser with limitations set forth as under:

The customer must obtain a return authorization supplied by the customer service team of Olive Health Care. The product must be returned adequately packed so as to avoid damages during transit, with copy of original sales receipt and copy of the RMA authorization

The customer must pay freight charges to and from Olive Health Care manufacturing unit. Olive Health Care does not warrant any damage charges caused during transit. Customer shall be responsible for detailed inspection before signing the BOL

This warranty shall not apply to product(s)/components that have been subject to misuse, negligence, modification, alteration, lack of routine maintenance and/or misapplication, or products damaged due to servicing other than Olive Health Care representatives

This warranty shall not apply to components or parts not manufactured by Olive Health Care. Olive Health Care will pass on the original manufacturer's warranty to the original purchaser. Please consult the service team for warranty details of such components.

UNLESS EXPRESSLY SET FORTH, THERE SHALL BE NO OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, OF FITNESS AND/OR MERCHANTABILITY OR ANY OTHER WARRANTIES IMPLIED BY CUTOM,USAGE OR COURSE OF DEALING

Olive Health Care's liability in this agreement is limited to repair/ replacement of any product or component purchased by the original purchaser. Olive Health Care disclaims responsibility for any incidental or consequential damages claimed to have arisen from any defect in workmanship or materials. No representative of Olive Health Care has the authority to change or enlarge the above warranty agreement.

For further support,  
Contact customer support team:  
support@oliveendoscopy.com  
Landline: 0484 403 8810