

OLIVE TRANSPORTATION CART



Read and understand all of the instructions and safety information in this manual before operating this product.



Yellow triangle with exclamation mark indicates potential minor injury to personnel or damage to equipment



Yellow triangle with lightning bolt indicates possible electric shock hazard

USER ATTENTION

All employees who will use the Olive transportation Cart must be briefed in its proper use and operation before using it. All employees who will utilize the Olive transportation Cart should be aware that reasonable precautions must be taken to ensure patient safety and the transportation cart's performance.

This product should only be used for the purpose for which it was created. Using this device in a way that is not recommended by Olive Health Care may void the equipment manufacturer's warranty. Olive Health Care disclaims all responsibility for the effects of using this product for purposes other than those for which it was intended. Misuse or alteration of a product can be dangerous. Olive Health Care disclaims all obligation for the results of product modifications, as well as the repercussions of using this product in combination with other products.



For the purpose of safety, wear appropriate PPE kit: mask, gloves, eye protection and gown while transporting endoscopes using Olive transportation Cart



Use of unqualified persons may lead to injury to personnel and/or damage to equipment. Personnel using the Olive transportation cart must be educated on best practices, use, and maintenance of the product to avoid injury or damage. Repair and adjustments should be attempted by persons qualified as representatives of the manufacturer

PRODUCT DESCRIPTION

Olive Endoscope Transportation Cart ensures ease of handling scopes. It considerably prevents cross-contamination and risk of scope damage during transit and cleaning. The large tray offers ample space for coiling the endoscopes in large hoops as per the SGNA endoscope transport guidelines.

INTENDED USE

The Olive Transportation Cart is designed to transport flexible endoscopes from the procedure room to the storage cabinet for reprocessing and clean endoscopes back to the procedure room.

GENERAL SPECIFICATIONS

- Stainless Steel Body
- Large and sturdy Trays to hold endoscope and cleaning fluids
- Multi-point tray grip
- Long-lasting build
- Leak resistant and anti-breakage
- Dimensions:
 - Standard cart size: 640mm x 650mm x 900 mm
 - Standard tray size: 640mm x 550mm x 150mm
- Weight: 25 kg
- Additional trays available for purchase on request

LIST OF PARTS

Item Name	Model No.	Units
Body	OTCSSB01	1
Trays	OTCFT05	5
Wheels	OTCPW04	4



MAINTENANCE AND BEST PRACTICES

CAUTION: The user is responsible for thoroughly cleaning and maintaining the transportation cart to ensure optimum functioning.

1. Do NOT coil the insertion tube or the universal cord of the endoscope with a diameter of less than 12 cm
2. Clean and disinfect the trays thoroughly on a regular cleaning protocol, or when the sinks becomes soiled.

CLEANING, DISINFECTION & CARE

Cleaning the Trays

Do not use abrasives, such as steel wool, wire brushes or scrapers on the stainless steel and PMMA fiber surface.

CAUTION: For plastic surfaces and acrylic plastic (PMMA Fiber) components, use a hospital-approved disinfectant. Avoid disinfectants that have glycol ethers as an ingredient. Do not use alcohol or ammonia.



Cleaning Stainless Steel Body of Transportation Cart

1. Remove dirt, blood, and other organic soil.
2. Always clean the trays prior to disinfection to avoid compromising the disinfectant's effectiveness.
3. Dust first, then remove any stubborn deposits with a damp sponge.
4. Using a soft cloth, clean with a non-abrasive, neutral liquid detergent at a 10-20% solution. Use a stronger concentration in the presence of excessive dirt.
5. Lightly rinse before disinfection, being careful to minimize moisture in the sinks.
6. Avoid using bleach for cleaning or disinfecting your sink's surface, as bleach causes permanent damage through oxidation and staining.

Given below is a list of materials/ agents that are approved to be used to clean Body of Olive Transportation Cart:

1. Soft, lint-free cloth
2. Mild detergents
3. White vinegar in a spray bottle
4. Sodium bicarbonate (Baking Soda)
5. Cleaners approved for use on Stainless Steel/Galvanized Iron
6. Non-abrasive cleaning pads
7. Isopropyl alcohol
8. Soft bristle brushes
9. Distilled water (pH rating 7.0) solely or with mild detergent

10. Hospital-grade non-bleach detergent

 CLEANING MATERIALS TO AVOID	 CLEANING AGENTS TO AVOID
Abrasive pearls	Hard water (water with a pH above 7.0)
Scrapers (metal or plastic)	Hydrochloric Acid
Steel wool	Steam or high pressure water on PMMA tray
Wire brushes	Bleach/ any compounds containing chlorine or sodium hypochlorite, or ammonium chloride salts

*Steam or high pressure water may be used to clean stainless steel body

REPAIRS AND REPLACEMENT

Please contact the Olive Health Care support team if you have any concerns with component replacement or repairs

WARRANTY

Olive Health Care warrants to the original purchaser of the Olive Transportation Cart that defective products/components shall be repaired/replaced by Olive Health care, so long as owned by the original purchaser with limitations set forth as under:

The customer must obtain a return authorization supplied by the customer service team of Olive Health Care. The product must be returned adequately packed so as to avoid damages during transit, with copy of original sales receipt and copy of the RMA authorization

The customer must pay freight charges to and from Olive Health Care manufacturing unit. Olive Health Care does not warrant any damage charges caused during transit. Customer shall be responsible for detailed inspection before signing the BOL

This warranty shall not apply to product(s)/components that have been subject to misuse, negligence, modification, alteration, lack of routine maintenance and/or misapplication, or products damaged due to servicing other than Olive Health Care representatives

This warranty shall not apply to components or parts not manufactured by Olive Health Care. Olive Health Care will pass on the original manufacturer's warranty to the original purchaser. Please consult the service team for warranty details of such components.

UNLESS EXPRESSLY SET FORTH, THERE SHALL BE NO OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, OF FITNESS AND/OR MERCHANTABILITY OR ANY OTHER WARRANTIES IMPLIED BY CUTOM,USAGE OR COURSE OF DEALING

Olive Health Care's liability in this agreement is limited to repair/ replacement of any product or component purchased by the original purchaser. Olive Health Care disclaims responsibility for any incidental or consequential damages claimed to have arisen from any defect in workmanship or materials. No representative of Olive Health Care has the authority to change or enlarge the above warranty agreement.

For further support,
 Contact customer support team:
support@oliveendoscopy.com
 Landline: 0484 403 8810